

Moving the *DST* Indoors

The *DST* is equipped with four retractable wheels for easy moving. In order to move the *DST*, carry out the following steps:

1. Lower the *DST* to the flat position.



2. Unplug the *DST* socket from the wall outlet.

3. Fold the front ramp 180 degrees.



4. Pull the front and rear handles. Now the *DST* is on wheels and can be easily moved.



Front handle

(The rear handle is at the other end of the *DST*).

5. After positioning the *DST* in its new location, lower the two handles.

6. Unfold the front ramp.